




Is your handpiece properly cared for?



**Specification**

- Number of handpieces per cycle ... Up to 4 handpieces.
- Minimum cycle time ... Air turbine : 15 sec.  
Straight and contra angle handpiece ( Light mode ) : 85 sec.  
( Heavy mode ) : 145 sec.
- Required air pressure... 0.4-0.6MPa
- Included couplings... For straight and contra angle handpiece 2pc.  
For chuck nozzle 1pc.
- Oil bottle capacity ... 500ml
- Power supply...100VAC-240VAC 50/60Hz
- Power consumption...30VA
- Dimensions...W280 x D275 x H308mm
- Net weight...5.5Kg (Standard spec)

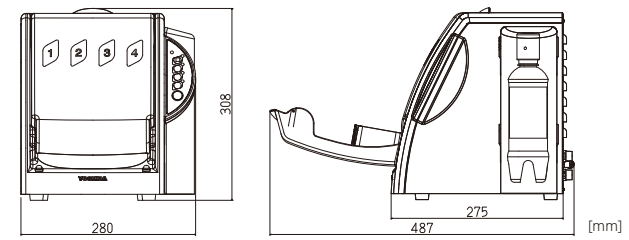

**Optional items**


- Oil bottle 500ml
- Adapter for Air turbine (ST)
- Adapter for Air turbine (LUX)
- Adapter for Straight and contra angle handpiece
- Adapter for Air scaler(LUX)
- Adapter for Air scaler (ST)

Distributor

**YOSHIDA**  
THE YOSHIDA DENTAL MFG.CO., LTD.  
Address 1-3-6 Kotobashi, Sumida-ku Tokyo, Japan ZIP 130-8516



Proper care requires lubrication between patients prior to the sterilization.

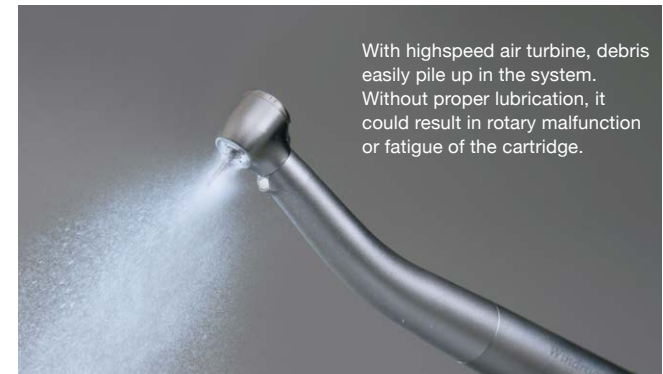

With highspeed air turbine, debris easily pile up in the system. Without proper lubrication, it could result in rotary malfunction or fatigue of the cartridge.

Lubrication before sterilization between patients is essential to the longevity of the cartridge. FRESHCARE PLUS assists you maintaining your handpieces properly and efficiently.

**Cleaning** Wipe down the surface of the handpiece after use.

Min. **15** sec.  
per air turbine!

Handpiece automatic maintenance system  
**FRESH CARE PLUS**

**Lubrication**

**Sterilization** Sterilize the handpiece.

**Storage** Store the handpiece in the sterilization bag until next use.

Handpiece automatic maintenance system  
**FRESH CARE PLUS**

# Automatic lubrication system for simple and proper maintenance to protect your handpiece.

## Holds up to 4 handpieces



### Chuck nozzle

The chuck of handpiece can be lubricated by attaching a chuck nozzle (included).



### Oil filter

Minimize the scattering of lubricant oil.



### Oil bottle


One oil bottle can lubricate approx. 1300 handpieces.



Handpiece automatic maintenance system

**FRESH CARE PLUS**

## Easy operation

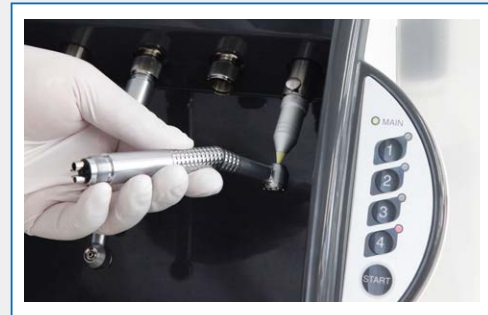

**STEP 1**

Lubricate the chuck.



**STEP 2**

Set the handpieces and close the front cover.



**STEP 3**

Select the mode and handpiece to lubricate and press START. Oil and compressed air will run through the entire system.